



PELVIC ASSISTANT


SURGIASSIST UTERINE POSITIONER

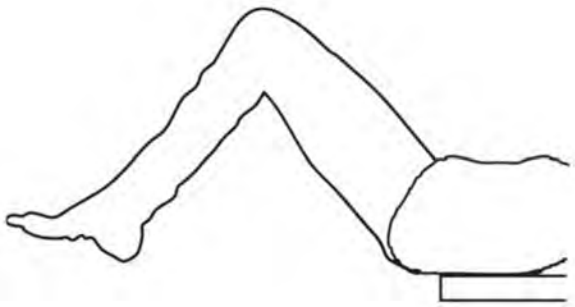

Buttocks OVER
edge of OR table

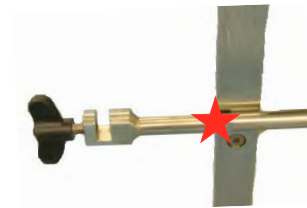

Notch in line with
end of table

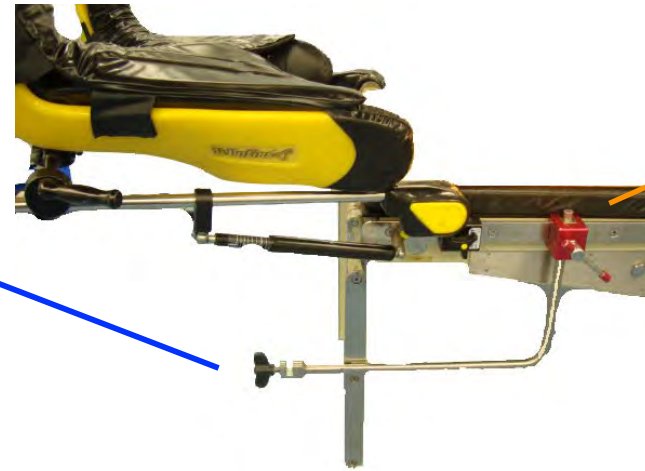

Left side rail



Top level
with clamp

1. Position Patient

2. Table Frame (non-sterile)



3. Sterile Assembly



Patient Midline

Tension Clamp
(sterile) into
Table Frame slot



OR technician
locks Table
Frame Screw
(non-sterile)




4. Position Holding Rod at Patient Midline



Ensure attachment is firm

5. Attach Uterine Manipulator

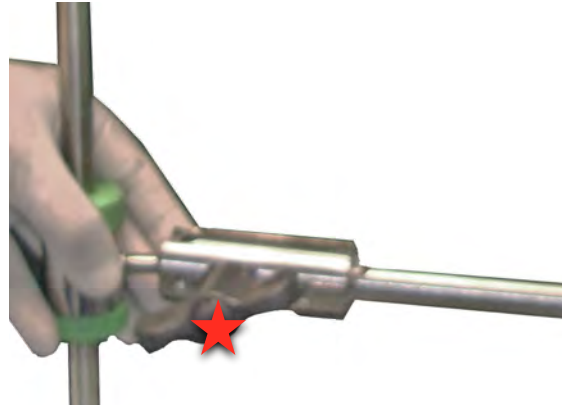


6. Adjust Tension



You must push handle down for anteversion



Rod will slide down inside Ball Joint

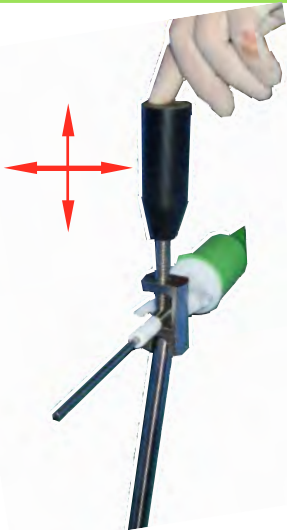


Push handle up, down, left & right

7. Manipulating Uterus



Gentle inward pressure optimises visualisation

8. Colpotomy

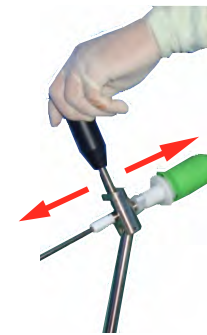

Simply flexing pivot joint will not achieve anteversion.

9. Anteverting