

## 1. Check Balloon inflates & deflates




## 2. Slide deflated Balloon over Colpotomizer

Approx 2cm below 20mm margin

Inflation tube should point down the Colpotomizer without doubling back over the Balloon




## 3. Place deflated Balloon ready for use

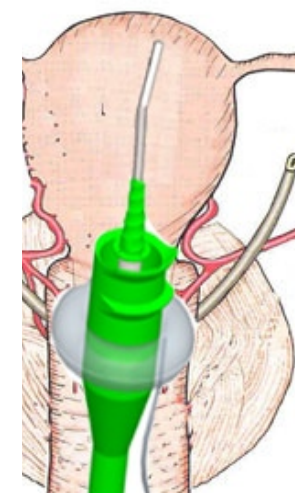

Inflate Balloon just enough to restore pneumoperitoneum if there is a leak during colpotomy or cuff closure

Never inflate more than necessary

## 4. Inflate just enough to stop gas leak

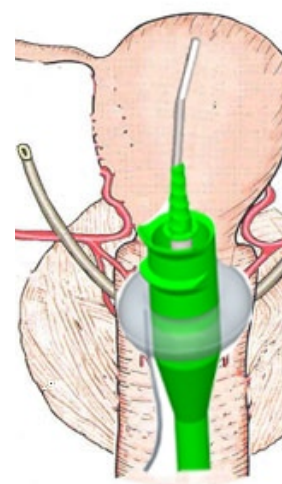

Deflate Balloon before rotating Colpotomizer

Reinflate if necessary when Colpotomizer is in new position

## 5. Deflate to rotate Colpotomizer