

1. Check Balloon inflates & deflates



2. Slide deflated Balloon over Colpotomizer

Approx 2cm below 20mm margin

Inflation tube should point down the Colpotomizer without doubling back over the Balloon




3. Place deflated Balloon ready for use



Inflate Balloon just enough to restore pneumoperitoneum if there is a leak during colpotomy or cuff closure

Never inflate more than necessary

4. Inflate just enough to stop gas leak

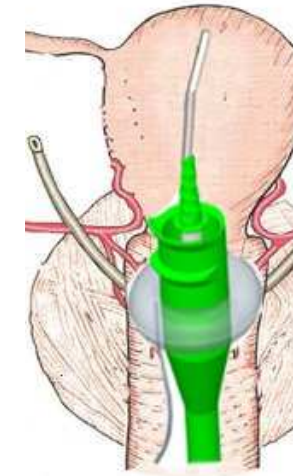

Deflate Balloon before rotating Colpotomizer

Reinflate if necessary when Colpotomizer is in new position

5. Deflate to rotate Colpotomizer